


Hand piece




Disconnect



Hole in yellow circle

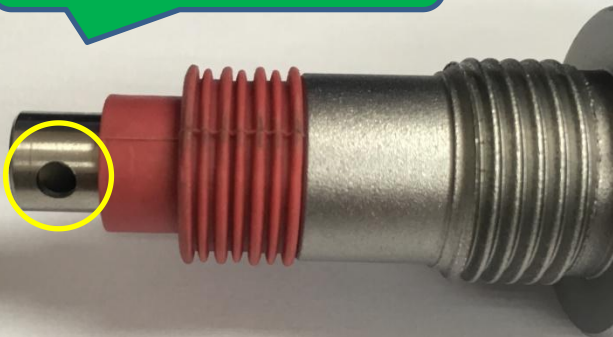

Make screwdriver through the hole

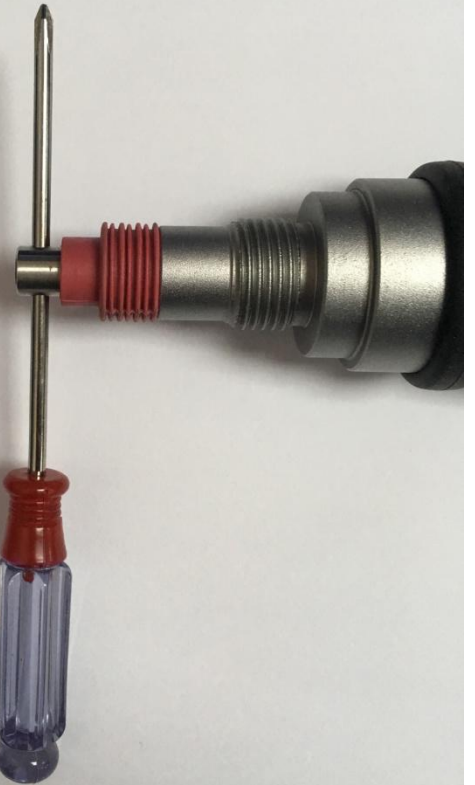

Pull screwdriver by hand



The pipe is pulled  
out of hand piece





You can find cleaning brush in white hand piece box




This is kind of lubricated oil you might find in store in your country, we can also replace it with alcohol

Spray oil inside the pipe



Clean the pipe , metal dirt  
might be hard to clean  
fully ,cleaning it a few times  
is a must.

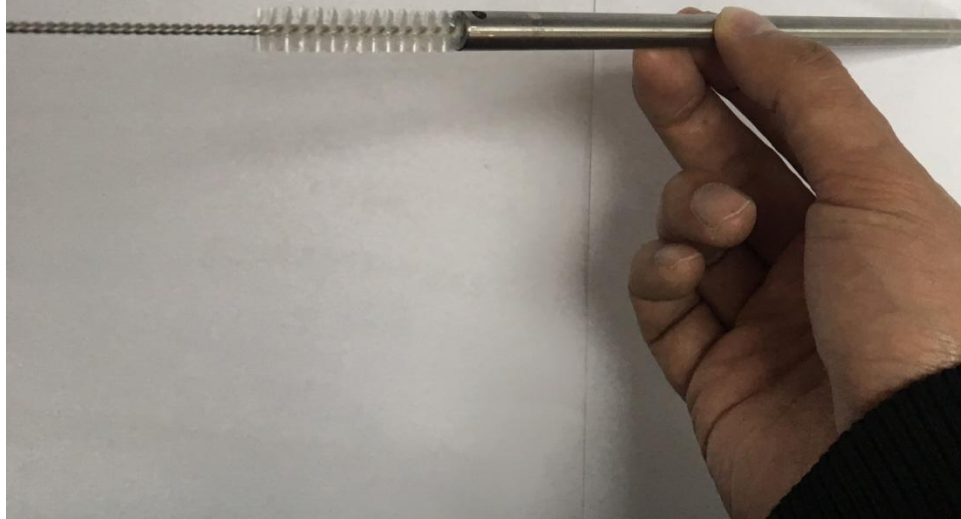



Clean the inner surface  
by dustless cloth



This is tube oil we are using now



You can purchase this 0W-20 oil with substitute of tube oil if you are not able to find tube oil in your country

## 0W-20 Motor Oil

Auto & Tires > Oils and Fluids >



\$27.47 18.0¢ / fl oz Pickup Only

Castrol EDGE 0W-20 Full Synthetic Motor Oil, 5 QT

★★★★★ (23)

Free store pickup today

Put oil around the  
bullet then clean bullet  
with dustless cloth



Put pipe and bullet into hand piece after  
cleanness, Do not install by wrong direction



Hand piece

